




PATIENT Bed for VIP ROOM HEB 904



Blue (Pantone 7682C)		Brown (Pantone 729C)		Red (Pantone 195C)		Orange (Pantone 715C)	
-------------------------	---	-------------------------	---	-----------------------	---	--------------------------	---

Supervisor Control Panel

The supervisor nurse control panel has clear graphic design for friendly using. Besides the function operation, lock / unlock, it contains 5 memory position - chair position, examination position, CPR, anti-shock and egress position.

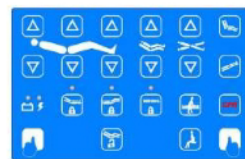

ACP04



Chair position

The chair position help the patient prevent respiratory complications and it will offer a better shape using environment for reading and dining.



Egress position

The egress position will lowering the bed height to min. The mattress platform will become 90° (back: 90° and feet: 0°).



Anti-shock position

The anti-shock position will lowering the back section and foot section, then the bed will be Trendelenbug position.



CPR position

The CPR position will bring the total bed to horizontal position in any position.



Examination position

The examination position will raise up the bed to max height and mattress platform down to the horizontal. It will help nurse staff to monitor and threat patient.



Action Button

The Action button safety concept protect the patient against injury from unexpected touch from nurse control panel. The all functions on nurse control panel should work with Action button before operation.

Purpose	General Ward/ VIP Ward
Dimension (platform & overall)	<ul style="list-style-type: none"> • 800 x 1920 mm (1000 x 2140 mm) • 860 x 1920 mm (1010 x 2140 mm) • 910 x 1920 mm (1060 x 2140 mm)
Adjustable height	<ul style="list-style-type: none"> • 440~750 mm (steel platform) • 450~760 mm (PP platform)
Back adjustment	0~70°
Knee adjustment	0~40°
Trendelenburg & Rev. Trendelenburg	+12° / -12°
Calf adjustment	10 steps
Auto-regression	Back/ buttock:10 cm/ 2 cm
Side rail specification	New tuck-away side rail with less trap and better protection (IEC 60601-2-52)
PP material	With gas-spring assistance and store compartment
Manual CPR (back)	Option
Control system	Low voltage motor system (4)
Recharge battery	Standard (30 times use after full charge)
Build in control unit	Panel 1 (inner x 2, outer x 2)
Nursing control panel	ACP 04
Urinal hook	Standard (both sides with 3 hooks)
IV pole and socket	1 units IV pole with 4 units socket
Sleeping surface	4 sections; movable powder coating steel pan with ventilation holes and body strap holder
Head/ foot board	PP material, detachable for emergency
Caster	125 mm
Locking system	Central locking system
X-ray cassette	Option
Under bed light system	Option
Lifting pole and socket	Available both sides of back
Bed extension	Option
Weight scale	Option
Loading capacity	Max. 250 kg

- Tolerance of dimension is about 5%.
- We reserve the right to change specification without notice